

Please read the entire instructions thoroughly before use!




1. Use a new needle each time. Remove the protective seal.



2. Wipe the connection junction with an alcohol swab before attaching the needle to the insulin pen, and screw it until locked.



3. Remove the outer needle cap. Keep it for later use.

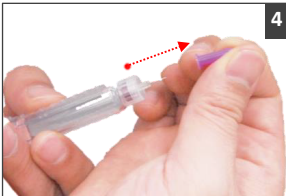

4. Remove and discard the inner cap.



5. To ensure you receive the full dose, inject the insulin slowly, according to your physician/medical professional's instructions.



6. Recap the used needle with the outer needle cap and unscrew the needle. Dispose of it in a sharps container.

HARMOFINE® Insulin pen needle fits all major brands of insulin pens.

Always consult your physician/medical professional to determine which needle is best for you.

WARNING

1. Do not use the needle if the protective seal is damaged to prevent infection or if the needle appears bent or damaged.
2. Pen needles are single-use only. Do not reuse or share needles to avoid serious risk of infection. Re-using needles will also make them blunt; it may hurt and cause bruising during subsequent injections.
3. Do not wiggle the needle during injection and withdrawal. It may cause the needle to bend or break.
4. Do not attempt to remove the needle without the outer needle cap in place.
5. Do not discard used needles in regular rubbish bins. Used needles must be disposed of in a sharps container. When the sharps container is full, seal it before disposal.


CAUTION

1. Prime the pen according to the manufacturer's instructions before each injection and check the insulin flow to ensure the pen and needle are working properly.
2. Follow your physician/healthcare professional's instructions and change your injection site regularly to prevent hardening of the skin tissue.
3. Store the needle in a cool, dry place and keep them away from children.



Manufactured for:
HARMONIUM PHARMA s.r.l.
Via Arona 6
20149 Milano - ITALY
info@harmonium-pharma.com

harmonium-pharma.com

 **CHIRANA T. Injecta, a.s.**
Nám. Dr. Schweitzera 194
916 01 Stará Turá,
Slovak Republic

www.t-injecta.sk

 2460